Instruction For Use

1.1 Language Selection

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Currently, this manual is available in Korean and English. Language selection is based on the overall content of countries with high usage and import potential. If a request or need to add a language is made, it will be reflected in the user manual.

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(Dressing, absorbable, wound, biological origin)

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◆ Dressing, absorbable, wound, biological origin

This product is a porous foam-type wound dressing that absorbs blood and exudate from the wound site, forming a physical barrier to prevent contamination and protect the wound, while also enabling hemostasis through physical action. In addition, it maintains a moist environment at the affected area, protecting the wound and providing conditions that promote wound healing.

◆ INTENDED USE

This is a bio-derived absorbable wound dressing intended for use on open wounds with heavy exudate and delayed healing, such as surgical wounds (including trauma and postoperative sites) and partial- or full-thickness wounds. (Excluding third-degree burns.) It is used to absorb blood and exudate from the wound site, prevent contamination, protect the wound, and achieve hemostasis through physical action.

◆ METHOD OF USE

- 1) Preparation Before Use
 - Check the packaging for damage and verify the expiration date before use.
 - Check the product and packaging for any abnormalities or damage.
 - Do not use if the product has been opened or if the packaging is damaged.
 - The instructions for use must be read carefully before use.
- 2) Instructions for Use
 - Before applying the product, clean the application site using sterile saline.
 - Remove the packaging step by step and take out the product.
 - Apply the product to the wound or area requiring hemostasis. If necessary, cut the product to fit the wound size and apply with gentle pressure. Secure with a secondary dressing, such as a bandage, if additional pressure or fixation is needed.
 - Remove the product within 24 hours, according to the healthcare professional's judgment. Replace with a new product as needed, depending on the amount of wound exudate. When applying a new product, clean the wound site with sterile saline before use.
- 3) Management Method After Use
 - **This product is a sterile, single-use medical device and must not be reused.**
 - Even if the product has not been used, once opened, it must be disposed of according to proper procedures.

◆ PRECAUTIONS FOR USE

- 1) Warnings
 - This product must be used under the guidance of a healthcare professional and must not be used for any purpose other than its intended use.
 - Do not use this product in individuals with hypersensitivity to its raw materials (bovine-derived gelatin, kaolin)
- 2) General Precautions
 - Before using this product, read and understand all instructions for use and precautions for use. Failure to follow the instructions and precautions may result in infection or adverse effects.
 - Do not use this product on patients with medical conditions for which its use is not appropriate.
 - Always check the expiration date before use. Do not use the product if the expiration date has passed.
 - Do not use the product if the packaging is damaged or has been opened prior to use.
 - This product is a sterile, single-use medical device and must not be resterilized or reused.
- 3) Application Precautions
 - This product is designed to degrade by 90% within 24 hours. It may be used once or replaced with a new product, but must not be used for more than 24 hours.
 - When applying a new product, clean the wound site with sterile saline to ensure complete removal of kaolin released from the previous product.

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◆ COMPLAINTS

Upon receiving any complaints or complaints regarding the quality, identity, durability, reliability, validity, and/or performance of the product, the user should notify manufacturer or their local representative. Furthermore, if the instrument is malfunctioning or suspected to be malfunctioning, it should be immediately notified to manufacturer or its local representative. If products of manufacturer have been improperly operated or have caused or contributed to the patient's serious injury or death, contact its local representative or manufacturer by phone, fax, or document as soon as possible.

Please fill out the following when receiving all complaints.

- Manufacturing number of components (Lot. Number)
- Product name (Part name)
- Part.No
- Name and address to be contacted
- Explain the cause of the complaint in detail

For additional information or complaints, please contact:

※Please check the attached label for product and manufacturer information.

Manufacturer : Mantiz Co., LTD.
Tel : +82.53.252.5833
Fax : +82.53.252.5834